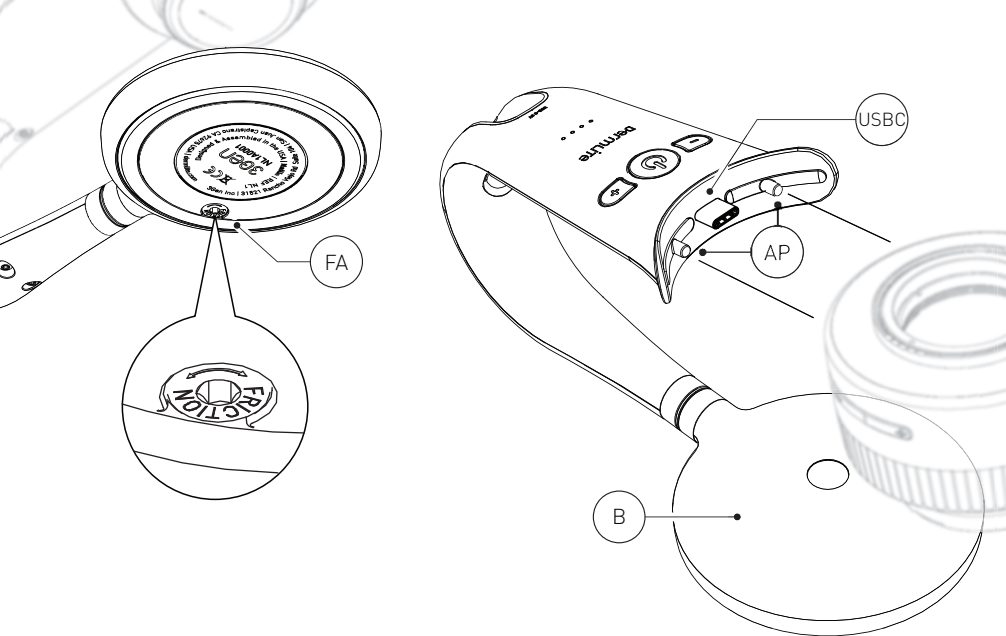
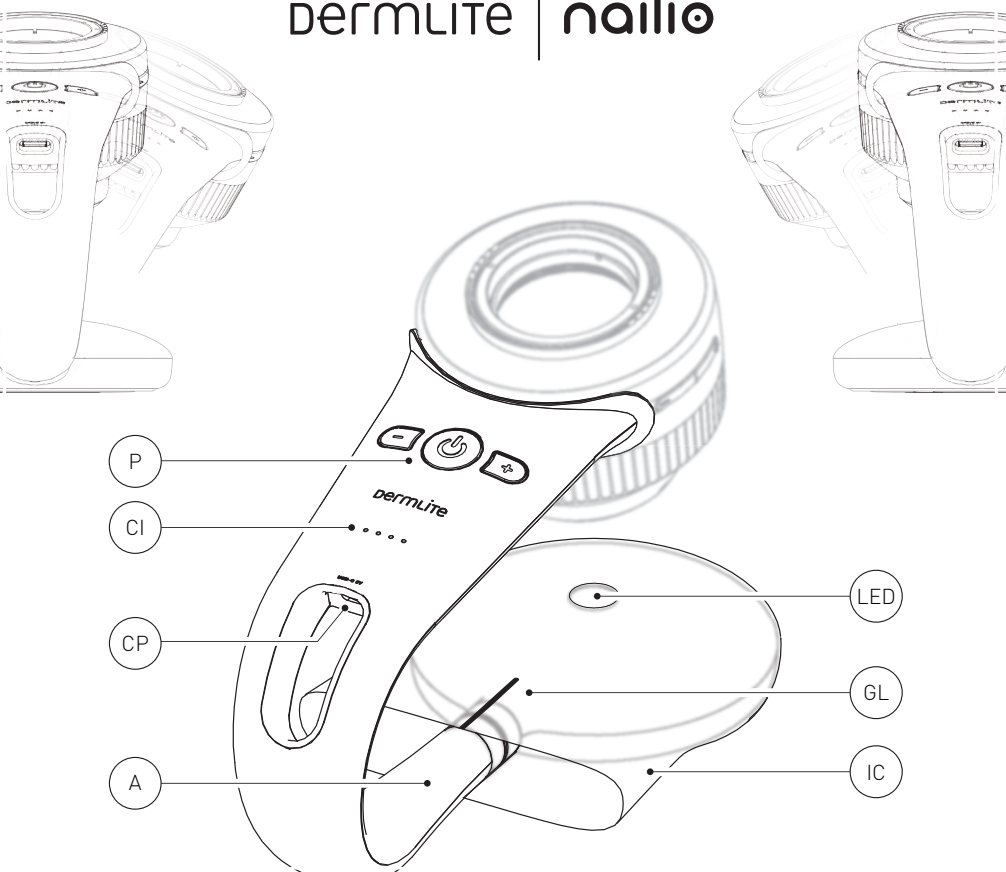
dermLite | nailio

Instructions

CAUTION: Prior to using this device, read these instructions and keep them in a safe place for future reference. Rx only. This device is designed for external examination only within professional healthcare facilities by medical professionals.

CAUTION: Do not use the device in fire or explosive risk area (e.g. oxygen-rich environment).

CAUTION: This product contains glass. Dropping or mishandling device may cause glass breakage. Be careful when handling broken glass; glass may be very sharp.

WARNING: This product can expose you to chemicals including methylene chloride and hexavalent chromium, which are known to the State of California to cause cancer or reproductive toxicity. For more information go to www.P65Warnings.ca.gov.

EMI Compatibility

This Product conforms to the electromagnetic emissions standard IEC 60601-1-2: 2014.

The EMISSIONS characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment. Portable and mobile RF communications equipment can affect medical electrical equipment.

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Use of accessories, transducers and cables other than those specified or

provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Directions for Use

Nailio® turns DermLite® Foto X (sold separately) into a nail examination device and provides enhanced visualization using red LED light.

CAUTION: Do not look directly into the red LED light. During use, the LED must be fully covered by a finger or toe. Patients must close their eyes during examinations.

CAUTION: Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the lights begin to flash.

IMPORTANT: Before initial use, please charge the internal lithium ion battery. See "Battery & Charging" for details.

Place the Nailio Base (B) on a table or another level work area. Attach Foto X to Nailio, making sure Nailio's two alignment pins (AP) match up with their mating alignment holes on Foto X. Slip the supplied IceCap® Cover (IC) over the Base (B), ensuring its printed guideline (GL) is parallel with Nailio's arm (A).

For best results, apply an immersion fluid such as ultrasound gel on your patient's nail. Place your patient's finger or toe over the LED's circular window (LED). Turn on Foto X (refer to Foto X instructions for use), then press the power button (P) to activate Nailio's red LED. Look through the Foto X lens and gently push down to make contact with the nail. Focus Foto X, if needed. Push the + or - buttons to choose between Nailio's 5 brightness levels. Adjust Foto X brightness as desired.

Tilt Nailio from side to side and/or ro-

tate your patient's nail for examination at various angles, ensuring the digit covers the red LED at all times. To allow for visual examination of the distal nail, Nailio may even be swiveled 90°. Swivel friction can be modified with a 2mm hex key (not included) via Friction Adjustment (FA). Press the power button to turn off the red LED. In order to save power, the device will turn off automatically after 3 minutes.

Battery & Charging

CAUTION: This device uses a specialized 3.7V 1400mAh lithium ion battery, which can only be purchased from 3Gen or from an authorized 3Gen dealer. Do not under any circumstances use another battery other than the one designed for this unit. Doing so could result in damage to the unit.

This device is equipped with a four-level Charge Indicator (CI). When the unit is turned on and fully charged, all four LEDs light up. When less than 75% of battery life remains, three LEDs are illuminated. If more than half the battery is drained, two LEDs are on, while a single LED indicates that less than 25% of battery capacity remains.

To charge your device, connect the supplied USB-C to USB cable to the Charging Port (CP) and any IEC 60950-1 compliant (5V) USB port.

The Charge Indicator (CI) pulses to indicate charging, with the number of pulsing LEDs corresponding with charge level.

When charging is complete, all four indicator LEDs illuminate. From a full charge, the device will allow approximately 3-8 hours of operation, depending on the selected brightness level.

After years of use, you may wish to replace the battery, which is only available directly from 3Gen or an authorized 3Gen dealer. To change the battery, contact 3Gen or your local 3Gen dealer.

Troubleshooting

Please check www.dermlite.com for the most current troubleshooting information. If your device requires servicing, visit www.dermlite.com/service or contact your local 3Gen dealer.

Care and Maintenance

Your device is designed for trouble-free operation. Never attempt to open the device for any reason. Repairs shall be made only by qualified service personnel.

Cleaning and Sterilization

The exterior of your device, except the optical parts, may be wiped clean with isopropyl alcohol (70% vol.). The lens should be treated as high-quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals. Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

The Nailio set includes:

Nailio examination device (1 polarized red LED), USB to USB-C charging cable, 5 IceCap covers.

Warranty

10 years for parts and labor. The rechargeable battery is warranted for 1 year. Serial number and company information are located on the Base.

Disposal

This device contains electronics and a lithium-ion battery that must be separated for disposal and may not be disposed of with general household waste. Please observe local disposal regulations.

Technical Description

Visit www.dermlite.com/technical/ or contact your local 3Gen dealer.

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Dermlite and 3Gen are registered trademarks of
©2019 by 3Gen, Inc. | All rights reserved. | REF A NL1-1701 | Patents/Trademarks: www.dermlite.com/patents
E. C. REP Ltd
3Gen, Inc. | 31521 Rancho Viejo Road | Suite 104 | San Juan Capistrano, CA 92675 | U.S.A.