dermLite<sup>®</sup>cam<sup>®</sup> Dermoscopy Camera

CAUTION: Prior to using DermLite cam, read these instructions and keep them in a safe place for future reference.

CAUTION: United States Federal law restricts this device to sale by or on the order of a physician.

CAUTION: The DermLite cam is designed for external examination only.

A: Power button  
B: Lens  
C: Flash  
D: Eyepiece  
E: Macro extender arm  
F: Battery cover  
G: SD card slot  
H: Lens cap  
I: Lens cleaning cloth  
J: Lens cap  
K: Lens cap  
L: Lens cap  
M: Lens cap  
N: Lens cap  
O: Lens cap  
P: Lens cap  
Q: Lens cap  
R: Lens cap  
S: Lens cap

DermLite cam is a digital camera designed to capture close up images of the skin under cross-polarized and non-polarized lighting. In cross-polarized mode, glare-free images are obtained by canceling the reflected light from the skin without immersion. In addition, non-polarized lighting may be used for traditional immersion fluid dermoscopy. In addition to non-polarized clinical (macro) images at the ideal distance, the camera's facelaple spacer may be removed for clearing.

Directions for Use

**CAUTION:** Do not look directly into the bright LED light. Patients must close their eyes during local examinations.

**IMPORTANT:** Before initial use, please charge your DCLcam for a minimum of 2 hours.

Charging

Prior to using the DCLcam, plug the included charger into an AC outlet. Then insert the cylindrical connector (A) into the power port (B) located on the bottom of the DCLcam body. The charging LED will turn red. Once the device is fully charged, the LED turns green. When operating the camera, a battery charge indicator appears on the screen.

After 500-1,000 charges, the battery's capacity will diminish and the battery may need to be replaced by an authorized 3Gen service provider. Contact 3Gen to arrange for service.

Using your DCLcam

After pressing the power button (C), DCLcam is ready to take images under cross-polarized light. Place the device directly onto the skin and press the shutter button (D) to take a photo and display it on screen. Images are sequentially numbered. To capture a non-polarized photograph, push the polarization button (X) to change modes, then press the shutter button.

To capture clinical images, slide the macro extender arm (E) forward until it locks, rotate the focus ring (M) to the macro position (P). Place the macro extender arm directly onto the skin and press the shutter button.

Facelaple Spacer

To remove the spacer, rotate it counter-clockwise and pull it off. This is the only part that may be stored on the SD card.

SD card (G)

All images are saved on the SD card. To remove from the SD card slot (I), simply push on the card. When inserting, make sure the 45° corner is facing up and forward.

**CAUTION:** If your DermLite cam is used for oil immersion imaging, place the oil only on the skin and then place the facelaple spacer on top of the oil. Do not allow oil to contact the metal DCLcam body, as this may damage the unit.

3gen<sup>®</sup>

dermLite, 3Gen, DermLite cam and DCLcam are registered trademarks of 3Gen, Inc. • 3Gen Facelaple Spacer Patent Info • 3Gen Skin Dermoscopy, CA 04/04 • U.S.A. Tel: +1 949 436 4316 • Fax: +1 949 436 2425 • Email: info@dermlite.com • www.dermlite.com

© 2005 3Gen, Inc. All rights reserved. • RNP 3/04/0007

Parents: www.dermlite.com/parents

ENGLISH

DANSK

DEUTSCH

ESPAÑOL

FRANÇAIS

## Instructions

DermLite cam<sup>®</sup> Dermoscopy Camera

CAUTION: Prior to using DermLite cam, read these instructions and keep them in a safe place for future reference.

CAUTION: United States Federal law restricts this device to sale by or on the order of a physician.

CAUTION: The DermLite cam is designed for external examination only.

DermLite cam is a digital camera designed to capture close up images of the skin under cross-polarized and non-polarized lighting. In cross-polarized mode, glare-free images are obtained by canceling the reflected light from the skin without immersion. In addition, non-polarized lighting may be used for traditional immersion fluid dermoscopy. In addition to non-polarized clinical (macro) images at the ideal distance, the camera's facelaple spacer may be removed for clearing.

Directions for Use

**CAUTION:** Do not look directly into the bright LED light. Patients must close their eyes during local examinations.

**IMPORTANT:** Before initial use, please charge your DCLcam for a minimum of 2 hours.

Charging

Prior to using the DCLcam, plug the included charger into an AC outlet. Then insert the cylindrical connector (A) into the power port (B) located on the bottom of the DCLcam body. The charging LED will turn red. Once the device is fully charged, the LED turns green. When operating the camera, a battery charge indicator appears on the screen.

After 500-1,000 charges, the battery's capacity will diminish and the battery may need to be replaced by an authorized 3Gen service provider. Contact 3Gen to arrange for service.

Using your DCLcam

After pressing the power button (C), DCLcam is ready to take images under cross-polarized light. Place the device directly onto the skin and press the shutter button (D) to take a photo and display it on screen. Images are sequentially numbered.

To capture a non-polarized photograph, push the polarization button (X) to change modes, then press the shutter button.

To capture clinical images, slide the macro extender arm (E) forward until it locks, rotate the focus ring (M) to the macro position (P). Place the macro extender arm directly onto the skin and press the shutter button.

Facelaple Spacer

To remove the spacer, rotate it counter-clockwise and pull it off. This is the only part that may be stored on the SD card.

SD card (G)

All images are saved on the SD card. To remove from the SD card slot (I), simply push on the card. When inserting, make sure the 45° corner is facing up and forward.

**CAUTION:** If your DermLite cam is used for oil immersion imaging, place the oil only on the skin and then place the facelaple spacer on top of the oil. Do not allow oil to contact the metal DCLcam body, as this may damage the unit.

| REF | Model Number |
|-----|--------------|
| 1   | dermLite     |
| 2   | dermLite     |
| 3   | dermLite     |
| 4   | dermLite     |
| 5   | dermLite     |
| 6   | dermLite     |
| 7   | dermLite     |
| 8   | dermLite     |
| 9   | dermLite     |
| 10  | dermLite     |
| 11  | dermLite     |
| 12  | dermLite     |
| 13  | dermLite     |
| 14  | dermLite     |
| 15  | dermLite     |
| 16  | dermLite     |
| 17  | dermLite     |
| 18  | dermLite     |
| 19  | dermLite     |
| 20  | dermLite     |
| 21  | dermLite     |
| 22  | dermLite     |
| 23  | dermLite     |
| 24  | dermLite     |
| 25  | dermLite     |
| 26  | dermLite     |
| 27  | dermLite     |
| 28  | dermLite     |
| 29  | dermLite     |
| 30  | dermLite     |
| 31  | dermLite     |
| 32  | dermLite     |
| 33  | dermLite     |
| 34  | dermLite     |
| 35  | dermLite     |
| 36  | dermLite     |
| 37  | dermLite     |
| 38  | dermLite     |
| 39  | dermLite     |
| 40  | dermLite     |
| 41  | dermLite     |
| 42  | dermLite     |
| 43  | dermLite     |
| 44  | dermLite     |
| 45  | dermLite     |
| 46  | dermLite     |
| 47  | dermLite     |
| 48  | dermLite     |
| 49  | dermLite     |
| 50  | dermLite     |
| 51  | dermLite     |
| 52  | dermLite     |
| 53  | dermLite     |
| 54  | dermLite     |
| 55  | dermLite     |
| 56  | dermLite     |
| 57  | dermLite     |
| 58  | dermLite     |
| 59  | dermLite     |
| 60  | dermLite     |
| 61  | dermLite     |
| 62  | dermLite     |
| 63  | dermLite     |
| 64  | dermLite     |
| 65  | dermLite     |
| 66  | dermLite     |
| 67  | dermLite     |
| 68  | dermLite     |
| 69  | dermLite     |
| 70  | dermLite     |
| 71  | dermLite     |
| 72  | dermLite     |
| 73  | dermLite     |
| 74  | dermLite     |
| 75  | dermLite     |
| 76  | dermLite     |
| 77  | dermLite     |
| 78  | dermLite     |
| 79  | dermLite     |
| 80  | dermLite     |
| 81  | dermLite     |
| 82  | dermLite     |
| 83  | dermLite     |
| 84  | dermLite     |
| 85  | dermLite     |
| 86  | dermLite     |
| 87  | dermLite     |
| 88  | dermLite     |
| 89  | dermLite     |
| 90  | dermLite     |
| 91  | dermLite     |
| 92  | dermLite     |
| 93  | dermLite     |
| 94  | dermLite     |
| 95  | dermLite     |
| 96  | dermLite     |
| 97  | dermLite     |
| 98  | dermLite     |
| 99  | dermLite     |
| 100 | dermLite     |

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Cleaning and Disinfection

The exterior of your DermLite may be cleaned or disinfected by wiping it with isopropyl alcohol (70% v/v). Do not use alcohol or disinfectants in the optical areas of the unit. Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

Cleaning the Lens and the Polarizers

The lens and the polarizing filters should be treated as high quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals.

DermLite cam components include: Camera, removable facelaple spacer with 10 mm scale, charger.

Serial number and camera information are located on the bottom of the DCLcam.

| REF | Modelnummer |
|-----|-------------|
| 1   | dermLite    |
| 2   | dermLite    |
| 3   | dermLite    |
| 4   | dermLite    |
| 5   | dermLite    |
| 6   | dermLite    |
| 7   | dermLite    |
| 8   | dermLite    |
| 9   | dermLite    |
| 10  | dermLite    |
| 11  | dermLite    |
| 12  | dermLite    |
| 13  | dermLite    |
| 14  | dermLite    |
| 15  | dermLite    |
| 16  | dermLite    |
| 17  | dermLite    |
| 18  | dermLite    |
| 19  | dermLite    |
| 20  | dermLite    |
| 21  | dermLite    |
| 22  | dermLite    |
| 23  | dermLite    |
| 24  | dermLite    |
| 25  | dermLite    |
| 26  | dermLite    |
| 27  | dermLite    |
| 28  | dermLite    |
| 29  | dermLite    |
| 30  | dermLite    |
| 31  | dermLite    |
| 32  | dermLite    |
| 33  | dermLite    |
| 34  | dermLite    |
| 35  | dermLite    |
| 36  | dermLite    |
| 37  | dermLite    |
| 38  | dermLite    |
| 39  | dermLite    |
| 40  | dermLite    |
| 41  | dermLite    |
| 42  | dermLite    |
| 43  | dermLite    |
| 44  | dermLite    |
| 45  | dermLite    |
| 46  | dermLite    |
| 47  | dermLite    |
| 48  | dermLite    |
| 49  | dermLite    |
| 50  | dermLite    |
| 51  | dermLite    |
| 52  | dermLite    |
| 53  | dermLite    |
| 54  | dermLite    |
| 55  | dermLite    |
| 56  | dermLite    |
| 57  | dermLite    |
| 58  | dermLite    |
| 59  | dermLite    |
| 60  | dermLite    |
| 61  | dermLite    |
| 62  | dermLite    |
| 63  | dermLite    |
| 64  | dermLite    |
| 65  | dermLite    |
| 66  | dermLite    |
| 67  | dermLite    |
| 68  | dermLite    |
| 69  | dermLite    |
| 70  | dermLite    |
| 71  | dermLite    |
| 72  | dermLite    |
| 73  | dermLite    |
| 74  | dermLite    |
| 75  | dermLite    |
| 76  | dermLite    |
| 77  | dermLite    |
| 78  | dermLite    |
| 79  | dermLite    |
| 80  | dermLite    |
| 81  | dermLite    |
| 82  | dermLite    |
| 83  | dermLite    |
| 84  | dermLite    |
| 85  | dermLite    |
| 86  | dermLite    |
| 87  | dermLite    |
| 88  | dermLite    |
| 89  | dermLite    |
| 90  | dermLite    |
| 91  | dermLite    |
| 92  | dermLite    |
| 93  | dermLite    |
| 94  | dermLite    |
| 95  | dermLite    |
| 96  | dermLite    |
| 97  | dermLite    |
| 98  | dermLite    |
| 99  | dermLite    |
| 100 | dermLite    |

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