

Treatment protocols

Step 1 Royal refreshing & peel



- 1) Before using HYDRA ROYAL PEEL, cleanse and degrease the skin with PREPEEL LOTION 1.
- 2) Take approximately 1.5ml of HYDRA ROYAL PEEL into a sterile syringe.
- 3) Apply the product in a thin layer onto the prepared skin topically and rub it in with intensive lifting massage movements, until partially absorbed.
- 4) Leave on the skin for 8-15min and then wash off with water and perform STEP II.

Step 2 Royal lift & hydration




Use Hydra Royal according to one of the following administration techniques:

- 1) microdepositional with a needle
- 2) retrograde with a needle or cannula
- 3) 5-point (Royal Lift Points)
- 4) 6-point (Royal Lift Six Points)
- 5) 10-point (Royal Skin Reconstruction)
- 6) Combined technique with additional use of Hydra Royal BOOSTER (Royal Hybrid)

Depth of administration:

- 1) deep layers of the dermis (needle)
- 2) at the border of the dermis and subcutaneous tissue (cannula)

Royal lift & hydration

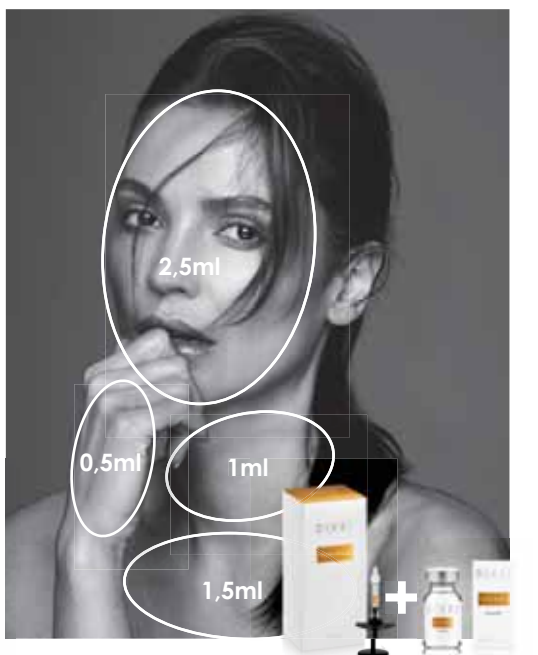

Step 3 Royal shine & soothing



- 1) Chill the mask in the fridge for 2 hours before use.
- 2) Apply to cleansed skin.
- 3) Place the gel side on the face according to the anatomical cutouts.
- 4) Remove the protective outer layer.
- 5) Leave on for 15-20 minutes and then remove.

Treatment effects with **Hydra Royal FAMI LY**

93%

PATIENT SATISFACTION

Most patients would recommend the treatment to a friend

87%

LASTING EFFECTS

Most patients positively rated the effects of the treatment series after 3 months

99%

SAFETY OF THE PROCEDURE

Most patients found the treatment procedure minimally invasive and complication-free



BEFORE

AFTER

Age of patient: **48 y.o.**
 Number of treatments: **3 treatments**

Improved hydration: **+89%**
 Smoothing of wrinkles: **+75%**
 Contour and oval improvement: **+34%**



BEFORE

AFTER

Age of patient: **41 y.o.**
 Number of treatments: **3 treatments**

Improved hydration: **+91%**
 Smoothing of wrinkles: **+80%**
 Contour and oval improvement: **+70%**



BEFORE

AFTER

Age of patient: **55 y.o.**
 Number of treatments: **3 treatments**

Improved hydration: **+69%**
 Smoothing of wrinkles: **+50%**
 Contour and oval improvement: **+60%**



BEFORE

AFTER

Age of patient: **45 y.o.**
 Number of treatments: **3 treatments**

Improved hydration: **+78%**
 Smoothing of wrinkles: **+73%**
 Contour and oval improvement: **+71%**



Hydra Royal

BEAUTY, WE MAKE IT POSSIBLE.





Dives Med GmbH
Schönhauser Allee 10-11,
10119 Berlin
Germany

divesmed.com